

# MEDIPHOT

## X-RAY PRODUCTS

### MEDIPHOT - Dev M

### DATASHEET

Developer for manual processing of X-Ray film, powder (2 parts), to mix 5 x 22,5 liters ready developer solution

#### Mix Instructions

1. Fill in about three quarters of water volume (30-40°C) being in accordance with the mix size of the unit.
2. Dissolve contents of the bags 1 and 2 completely with stirring.
3. Top up to the final volume in accordance with the mix size of the unit.

**Processing conditions** 18°C 5-6 min / 20°C 4-5 min / 22°C 3-4 min / 24°C 2-3 min

#### Ansatzvorschrift

1. Etwa ¾ des auf der Packung angegebenen Ansatzvolumens als Wasser mit 30-40°C vorlegen.
2. Inhalt der Packungsteile 1 und 2 unter ständigem Rühren lösen.
3. Auf der Packung angegebenes Endvolumen auffüllen, gut durchmischen.

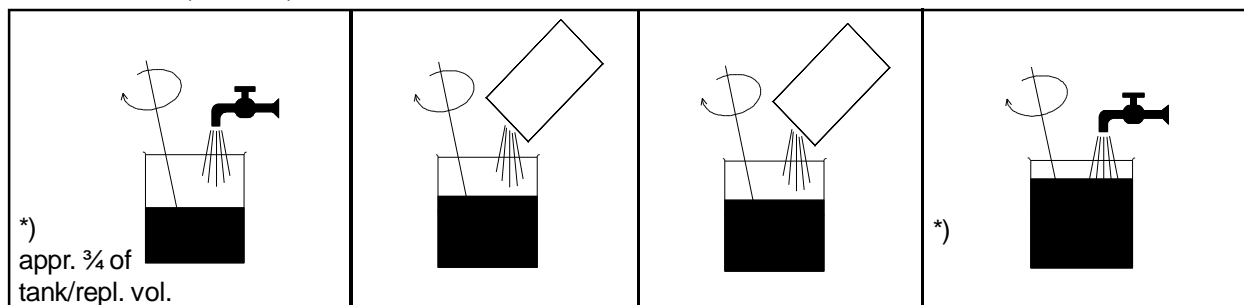
**Verarbeitungsbedingungen** 18°C 5-6 min / 20°C 4-5 min / 22°C 3-4 min / 24°C 2-3 min

#### Instruction de preparation

1. Remplir environ ¾ du volume de la cuve avec l'eau (30-40°C).
2. Vider les sachets 1 et 2 dans le cuve en agitant bien.
3. Completer le volume d'eau necessaire et bien melanger.

**Conditions de developpement** 18°C 5-6 min / 20°C 4-5 min / 22°C 3-4 min / 24°C 2-3 min

1. Water (30-40°C)      2. Powder Part 1      2. Powder Part 2      3. Fill up with water



\*) For mixing ready solution in tank 22,5 liters  
For mixing replenishment solution 18,5 liters

Processing Conditions:	Fixer Temperature		Time in Fixer	
		18°C	5-6 min	
	20°C	4-5 min		
	22°C	3-4 min		
	24°C	2-3 min		

**Replenishment Rate:** to process 1m<sup>2</sup> of film, under normal conditions, repl. rate appr. 300-400 ml

**Storage:** keep MEDIPHOT-Fix M in close PE-bag, storage temp. between 10-25°C

COLENTALABORTECHNIK  
Ges.m.b.H. & Co. KG  
Neunkirchner Strasse 117  
A-2700 Wiener Neustadt  
Tel.: +43-2622-28311-0  
Fax: +43-2622-28311-7  
E-mail: [office@colenta.at](mailto:office@colenta.at)  
<http://www.colenta.at>

COLENTALABORTECHNIK  
Ges.m.b.H. & Co. KG  
Ratzinger Weg 9  
D-94065 Waldkirchen  
Tel.: +49-8581-207-0  
Fax: +49-8581-207-34  
E-mail: [info@colenta.de](mailto:info@colenta.de)  
<http://www.colenta.de>

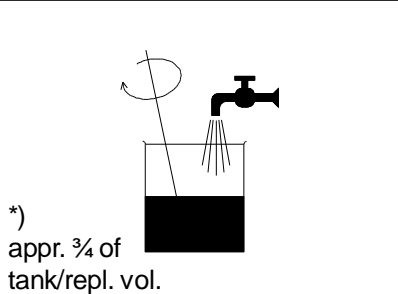
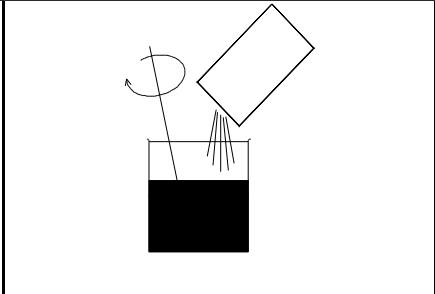
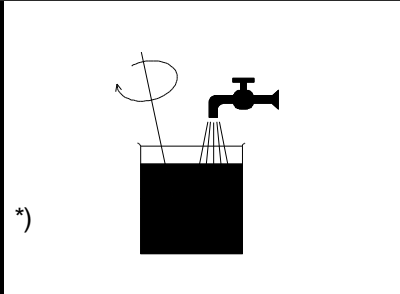
ACI-COLENTA  
INTERNATIONAL Div.  
Forge Close, Little End Rd,  
Eaton Socon, Cambs. UK  
Tel. +44-1480-211118  
Fax: +44-1480-215236  
EFax: +44/870/132 1352  
E-mail: [aci@patflaherty.com](mailto:aci@patflaherty.com)

### MEDIPHOT - Fix M

### DATASHEET

Fixer for manual processing of X-Ray film, powder (1part), to mix 5 x 22,5 liters ready fixer solution

<p><b>Mix Instructions</b></p> <ol style="list-style-type: none"> <li>1. Fill in about three quarters of water volume (30-40°C) being in accordance with the mix size of the unit.</li> <li>2. Dissolve contents of the bag completely with stirring.</li> <li>3. Top up to the final volume in accordance with the mix size of the unit.</li> </ol> <p><b>Processing conditions</b> <span style="float: right;"><b>20°C 4-5 min</b></span></p>
<p><b>Ansatzvorschrift</b></p> <ol style="list-style-type: none"> <li>1. Etwa <math>\frac{3}{4}</math> des auf der Packung angegebenen Ansatzvolumens als Wasser mit 30-40°C vorlegen.</li> <li>2. Inhalt der Packung unter ständigem Rühren lösen.</li> <li>3. Auf der Packung angegebenes Endvolumen auffüllen, gut durchmischen.</li> </ol> <p><b>Verarbeitungsbedingungen</b> <span style="float: right;"><b>20°C 4-5 min</b></span></p>
<p><b>Instruction de preparation</b></p> <ol style="list-style-type: none"> <li>1. Remplir environ <math>\frac{3}{4}</math> du volume de la cuve avec l'eau (30-40°C).</li> <li>2. Vider le sachet dans la cuve en agitant bien.</li> <li>3. Compléter le volume d'eau nécessaire et bien mélanger.</li> </ol> <p><b>Conditions de developpement</b> <span style="float: right;"><b>20°C 4-5 min</b></span></p>

1. Water (30-40°C)	2. Powder	3. Fill up with water
		

\*) For mixing ready solution in tank 22,5 liters  
For mixing replenishment solution 18,5 liters

Processing Conditions:	Fixer Temperature	Time in Fixer
	18°C	5-6 min
	20°C	4-5 min
	22°C	3-4 min
	24°C	2-3 min

**Replenishment Rate:** to process 1m<sup>2</sup> of film, under normal conditions, repl. rate appr. 600 ml

**Storage:** keep MEDIPHOT-Fix M in close PE-bag, storage temp. between 10-25°C

COLENTALABORTECHNIK  
Ges.m.b.H. & Co. KG  
Neunkirchner Strasse 117  
A-2700 Wiener Neustadt  
Tel.: +43-2622-28311-0  
Fax: +43-2622-28311-7  
E-mail: [office@colenta.at](mailto:office@colenta.at)  
<http://www.colenta.at>

COLENTALABORTECHNIK  
Ges.m.b.H. & Co. KG  
Ratzinger Weg 9  
D-94065 Waldkirchen  
Tel.: +49-8581-207-0  
Fax: +49-8581-207-34  
E-mail: [info@colenta.de](mailto:info@colenta.de)  
<http://www.colenta.de>

ACI-COLENTA  
INTERNATIONAL Div.  
Forge Close, Little End Rd,  
Eaton Socon, Cambs. UK  
Tel. +44-1480-211118  
Fax: +44-1480-215236  
EFax: +44/870/132 1352  
E-mail: [aci@patflaherty.com](mailto:aci@patflaherty.com)